

Echinococcus Granulosus Antibody Rapid Test (CE Ab)

Catalog No.: EVG0470

◇ INTENDED USE

The Echinococcus Granulosus Antibody Rapid Test is a rapid test for the detection of antibodies to Echinococcus

Granulosus (EGR Ab) in sheep's serum or plasma specimen.

Assay Time: 5-10 minutes

Specimen: serum, plasma

◇ PRINCIPLE

The Echinococcus Granulosus Antibody Rapid Test is based on sandwich lateral flow immunochromatographic assay. The test device has a testing window for the observation of assay running and result reading. The testing window has an invisible T (test) zone and a C (control) zone before running the assay. When the treated sample was applied into the sample hole on the device, the liquid will laterally flow through the surface of the test strip and react with the pre-coated EGR antigens. If there are EGR antibodies in the specimen, a visible T line will appear. The C line should always appear after a sample is applied, which indicates a valid result. By this means, the device can accurately indicate the presence of Echinococcus Granulosus antibodies (EGR Ab) in the specimen.

◇ REAGENTS AND MATERIALS

- Test devices, with disposable droppers
- Products Manual

◇ STORAGE AND STABILITY


The kit can be stored at room temperature (2-30°C). The test kit is stable through the expiration date (24 months) marked on the package label. DO NOT FREEZE. Do not store the test kit in direct sunlight.

◇ TEST PROCEDURE

- Collect sheep's fresh whole blood, get the serum or plasma specimen for use. Immediately apply the testing once collected the specimen. If not use right now, keep storage at 2-8°C. If the storage time more than 24hours, the specimen should be frozen at -20°C.
- Allow all materials, including specimen and test device, recover to 15-25°C before running the assay.
- Take out the test device from the foil pouch and place it horizontally.
- Place 3 drops (approx. 120µL) of serum or plasma specimen into the sample hole of the test card.

- Interpret the result at 10 minutes. Result after 15 minutes is considered as invalid.

◇ INTERPRETATION OF RESULTS

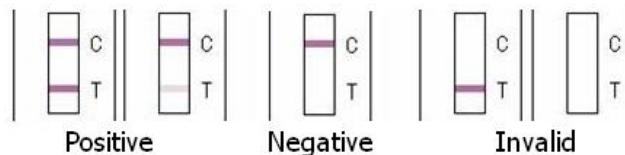

- **Positive (+):** The presence of both "C" line and zone "T" line, no matter T line is clear or vague.
- **Negative (-):** Only clear C line appear. No T line.
- **Invalid:** No colored line appears in C zone. No matter if T line appears.

◇ PRECAUTIONS

- All reagents must be at room temperature before running the assay.
- Do not remove test cassette from its pouch until immediately before use.
- Do not use the test beyond its expiration date.
- The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.
- All specimens are of potential infection. It must be strictly treated according to the rules and regulations by local states.

◇ LIMITATION

The Echinococcus Granulosus Antibody Rapid Test is for in vitro veterinary diagnosis use only. All result should be considered with other clinical information available with veterinarian. It is suggested to apply a further confirmative method when positive result was observed.



HANGZHOU EVEGEN BIOTECH CO., LTD.

NO.578, 20th Street, Qiantang District, Hangzhou, Zhejiang
310018 China

PN:20220916EVG0470